

Lone Star™ RETRACTOR SYSTEM


...better than another pair of hands™

Gynecology Kit


Self-retaining retractor system for a wide range of gynecologic surgical procedures

Benefits

- Essential components for key procedures:
 - Urinary incontinence procedures including slings
 - Pelvic organ prolapse repairs
 - Vaginal mesh revision/excision
 - Vesicovaginal fistula repairs
 - Rectovaginal fistula repairs
 - Perineoplasty
- Superior exposure and visualization
- Frees up OR personnel to perform other tasks
- Cost-effective
- Easy to use
- Easy to store, stock, pick
- Simplifies surgery
- Provides effective surgical site management with less clutter and better exposure

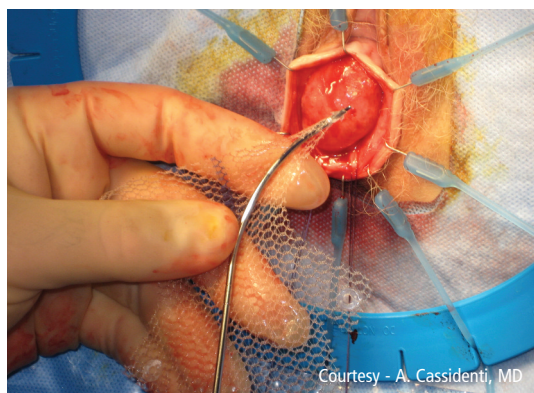

Courtesy - A. Cassidenti, MD

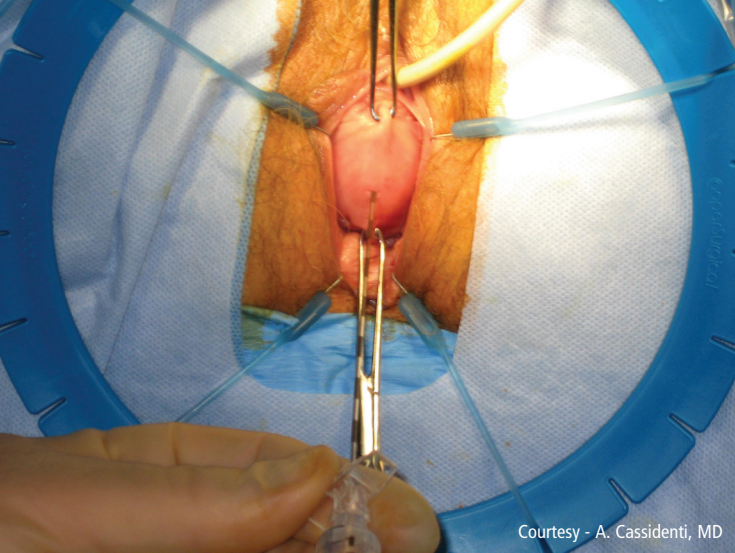

Courtesy - A. Cassidenti, MD

The Lone Star Retractor System facilitates mid-line incision of the anterior vaginal wall during cystocele repair.




Less Clutter.

Better Exposure.

Effective Surgical Site Management.

Ease and Precision: 3 Simple Steps

1. Place the Retractor Ring around the initial incision. Insert elastic stay hooks in tissue and retract in directly opposing quadrants. Slots on ring allow easy placement for equal and constant tension.
2. As dissection progresses, reposition hooks and add new ones as needed to achieve full, balanced retraction from any angle.
3. When closing, place stays along axis of the incision to create tension to facilitate precise staple or suture placement.

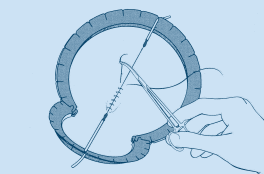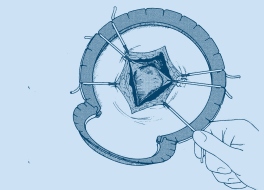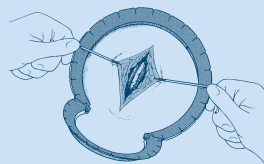

Lone Star Retractor System™ Gynecology Kit

Get convenience without added cost.

At last, surgical access the way it should be. Unparalleled exposure. Under your control. This unique retractor system delivers uncluttered access using counter-traction with small hooks. The result: a surgical site that you can easily adapt as the procedure progresses.

Each kit contains one large snowman disposable retractor (32.5 cm x 18.3 cm – Model #3304G), an eight-pack of 5 mm sharp hook elastic stays (model number 3311-8G) and two catheter clips for the Foley Catheter.

The cost is the same as if purchased separately, but the convenient packaging facilitates inventory management making it easy to store, pick and present to the field.

Ordering Information:

PRODUCT NUMBER	DESCRIPTION
3704	<p>Each Lone Star Retractor System Gynecology Kit contains:</p> <ul style="list-style-type: none"> • One large snowman disposable retractor ring, 32.5 cm x 18.3 cm – Model #3304G • One eight-pack of 5 mm sharp hook stays – Model #3311-8G • Two catheter clips

Each kit is packaged in a single-use sterile dispensing tray. Sold in boxes containing five kits.

CooperSurgical

CooperSurgical is the leading company dedicated to providing medical devices and procedure solutions that improve health care delivery to women regardless of clinical setting. Our company is fostering that position through expansion of its core businesses and introduction of advanced technology-based products which aid clinicians in the management and treatment of commonly seen conditions.

203.601.5200 | 800.243.2974 | coopersurgical.com | 95 Corporate Dr. | Trumbull, CT 06611
Also on YouTube at www.YouTube.com/CooperSurgical

Lone Star Retractor System™ is a trademark of CooperSurgical.
U.S. Patent Nos. 5,769,783; 5,785,649; 5,964,697.

© Copyright 2012 CooperSurgical, Inc

Form #81451 • 01/12